



Freezpoint™

INDICATIONS FOR USE

SKIN TAGS: Treatment Time 5-10 seconds



BEFORE TREATMENT



POST TREATMENT



DAY 7



DAY 14

AGE SPOTS: Treatment Time 5-10 seconds




BEFORE TREATMENT



DAY 14



DAY 28

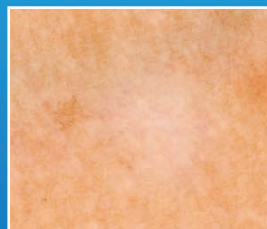

DAY 60



Freezpoint™

REORDER # CRY16



The number of procedures per unit is dependent on the time spent on each procedure. Treatment of different lesions during the same procedure is considered separate procedures. Retreatment or using a freeze-thaw-freeze cycle must account in total spray time.

**REORDER IS RECOMMENDED
AFTER THE 15TH USE.**

Bovie Medical Corporation
5115 Ulmerton Road • Clearwater, FL 33760
Ph 1-800-537-2790 • Fax 1-727-347-9144
sales@boviemed.com • www.boviemed.com

Rev. 0

TREATMENT SPRAY TIME

(seconds by approved lesion)

1	11	21
2	12	22
3	13	23
4	14	24
5	*15	25
6	16	26
7	17	27
8	18	28
9	19	29
10	20	30