

MICROKLEEN™ HS POWDER

High Suds Neutral Detergent



Micro-Scientific™ MicroKleen™ HS Powder High Suds Neutral Detergent is specially formulated for the manual cleaning of glass, plastic, rubber and metal surgical instruments and equipment. It removes blood, tissue and soil, and leaves instruments moving freely and spot-free.

- **Reduces likelihood of rust, tarnish or corrosion.**
- **Rinses spot & residue-free.**
- **Product safe and compatible with all metals.**

MicroKleen™ HS Powder



High Suds Neutral Detergent

Order No.	Size	Physical Properties	
B3	5 lbs. (2.27 kg)	Form:	Powder
		Color:	White
		Odor:	Odorless
		pH Concentration:	7.9
Uses		Temperature	
<input checked="" type="checkbox"/>	Presoak	Usage:	Room Temperature
<input checked="" type="checkbox"/>	Manual	Storage:	Cool, dry area
<input type="checkbox"/>	Ultrasonic		
<input type="checkbox"/>	Automatic Washer		
<input type="checkbox"/>	Automatic Endoscope Reprocessor		
<input type="checkbox"/>	Cart Washer		
Product Features		Consumer Benefits	
Rust Inhibitors:		Reduces likelihood of rust, tarnish or corrosion.	
Rinsing Agents:		Rinses spot & residue-free.	
Neutral pH:		Product safe and compatible with all metals.	
Concentrated:		Low cost per use.	
Bacteriostatic:		If stored properly, not favorable to bacterial growth.	
Environmentally Friendly:		Biodegradable, phosphate-free, VOC free formula.	
Procedures		Directions for Use	
Directions:		Wear appropriate personal protective equipment based on safe work practices.	
Presoak and Manual Cleaning:		Add 1 scoop (1/2 ounce) (14.8 mL) per 1 gallon (3.8 L) of warm water. Solution should be aspirated through lumens and channels of instruments. Discard solution after each use. Rinse thoroughly.	